



Exquisite Restoration™

Exquisite Restoration™ Instructions

Composite Fillings:

1. Isolate and prepare the tooth surface to be restored. Insert a proper matrix where indicated.
2. Use a cavity cleanser such as **Clean & Boost™** to make the cavity as clean as possible.
3. Apply an adhesive, such as **SURPASS®** Self-Etch Adhesive system or **SUPERB** Universal Adhesive, according to the manufacturer's instructions.
4. In Class I's and Class II's insert a thin layer (< 0.5 mm) of flowable composite (such as **Titan** Flowable Composite) over the entire pulpal floor up to and including the cavosurface margins. Activate the flowable composite according to the manufacturer's instructions.
5. Insert a 2 mm increment of the desired shade of **Exquisite Restoration™** and properly adapt to the cavity.
6. Light-cure for 20 seconds.
7. Continue inserting increments and light curing until cavity is slightly overfilled.
8. Finish and polish as usual.