



Dental Materials, Inc.

RE-GEN Pit & Fissure Sealant

REF 90662

Description:

RE-GEN Pit & Fissure Sealant is a filled resin sealant that provides the benefits of extended fluoride, calcium and phosphate release. It is used to aid in sealing the surface of a tooth to prevent decay while providing ions to promote remineralization.

Indications for use:

RE-GEN Pit and Fissure Sealant used for prophylactic sealing of pits and fissures.

Instructions for use:

1. Isolate and pumice the tooth surface that will be restored
2. Etch surface for 20 seconds with RE-GEN Cleanser
3. Apply thin coat of RE-GEN Pit & Fissure Sealant using applicator tip
4. Air thin
5. Light activate for 20 seconds

Storage

1. Protect from sunlight and excessive temperatures (<30°C/85°F)
2. Store at room temperature
3. Bring material to room temperature before use

Contradictions

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known methacrylate allergies. If prolonged contact with oral soft tissue occurs, flush with copious amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product with this patient.

Precautions and Warnings

1. Carefully read and understand all instructions prior to use.
2. RX Only
3. Clinician should wear UV protective eyewear when curing light is activated.
4. Verify flow through syringe prior to use on patient. If there is resistance, replace tip and check flow again.
5. Redirect overhead light to prevent premature polymerization.



Dental Materials, Inc.

RE-GEN Pit & Fissure Sealant

6. Avoid repeated contact with skin, may be sensitizing.
7. Resin may set in tip after a few hours. To remove clogging, always recap tip between each use.
8. Tips are disposable. To avoid cross contamination, DO NOT reuse tip.



WARNING

Causes serious eye irritation.

Wear eye/face protection. Prolonged or repeated skin contact may cause severe irritation.

FIRST AID:

IF IN EYES: Rinse cautiously with water for several minutes, also under eye lids. Remove contact lenses, if present and easy to do so. **IF INHALED:**

Move to fresh air and keep at a rest position comfortable for breathing. **IF**

INGESTED: Do not induce vomiting. Rinse mouth, drink plenty of water. Call a physician or Poison Control Center immediately.

Rx ONLY



Inter-Med, Inc.

2200 South St.,

Racine, WI 53404

Tel: +1-262-636-9755



Emergo Europe

Prinsessegracht 20

2514 AP The Hague

The Netherlands

91025-I-AP-ENG (1)